

ATLANTIS™ screw torque recommendations for ATLANTIS™ abutments

Manufacturer	Implant	Torque Ncm	Screwdriver	
BioHorizons	Internal 3.5, 4.5, 5.7	30	0.050" (1.25 mm) Hex screwdriver	
	Tapered Internal 3.5, 4.5, 5.7	30		
	External implant 3.5	30		
	External implant 4.0, 5.0	30		
Biomet 3i	Certain™ 3.25, 4.0, 4/3, 5.0, 5/4, 6.0	20	0.048" Hex screwdriver	
	Certain™ Prevail™ 3/4/3 (3.4), 4/5/4, 5/6/5, 4/3, 5/4, 6/5	20		
	Certain™ XP 4/5, 5/6	20		
	Miniplant™ 3.25	20		
	Osseotite™ 3.75, 4.0, 5.0, 6.0	20		
	Osseotite™ XP 4/5, 5/6	20		
Camlog	Screw-line Implant 3.3	20	1.28 mm Hex screwdriver	
	Screw-line / Root-line Implant 3.8, 4.3, 5.0, 6.0	20		
DENTSPLY Implants	ANKYLOS® C/X – C/ 3.5, 4.5, 5.5, 7.0 (non-indexed)	15	1.0 mm Hex screwdriver	
	ANKYLOS® C/X – /X 3.5, 4.5, 5.5, 7.0 (indexed)	15		
	OsseoSpeed™ EV 3.0 – Green	25	0.050" Hex screwdriver	
	OsseoSpeed™ EV 3.6 – Purple	25		
	OsseoSpeed™ EV 4.2 – Yellow	25		
	OsseoSpeed™ EV 4.8 – Blue	25		
	OsseoSpeed™ EV 5.4 – Brown	25		
	OsseoSpeed™ Profile EV 4.2 Yellow	25		
	OsseoSpeed™ Profile EV 4.8 Blue	25		
	OsseoSpeed™ / OsseoSpeed™ TX 3.0 – Yellow	15		
	OsseoSpeed™ / OsseoSpeed™ TX 3.5, 4.0 – Aqua	20		
	OsseoSpeed™ / OsseoSpeed™ TX 4.5, 5.0 – Lilac	25		
	OsseoSpeed™ 4.0 - Blue (Previous)	25		
	OsseoSpeed™ TX Profile 4.5, 5.0	25		
	XIVE® S 3.0	24	1.22 mm Hex screwdriver	
	XIVE® S 3.4, 3.8, 4.5, 5.5	24		
	Keystone Dental	PrimaConnex™ SD 3.3, 3.5, Genesis™ 3.8	30	Keystone Quad Driver
		PrimaConnex™ RD 4.0, 4.1, Genesis™ 4.5	30	
PrimaConnex™ WD 5.0, Genesis™ 5.5, 6.5		30		
Nobel Biocare	NobelActive™ 3.0	15	Nobel Biocare Unigrip	
	NobelActive™/NobelReplace™ Conical Connection NP 3.5	35		
	NobelActive™/NobelReplace™ Conical Connection RP 4.3, 5.0	35		
	NobelReplace™ NP 3.5	35		
	NobelReplace™ RP 4.3	35		
	NobelReplace™ WP 5.0	35		
	NobelReplace™ 6.0	35		
	Brånemark System™ NP 3.3	35		
	Brånemark System™ RP 3.75, 4.0	35		
Brånemark System™ WP 5.0, 5.5	35			

Please turn over

ATLANTIS™ screw torque recommendations for ATLANTIS™ abutments

Manufacturer	Implant	Torque Ncm	Screwdriver
Straumann	Bone Level 3.3 NC (Narrow CrossFit™)	35	SCS screwdriver
	Bone Level 4.1, 4.8 RC (Regular CrossFit™)	35	
	Standard/Standard Plus 4.8 RN (Regular Neck)	35	
	Standard/Standard Plus 4.8 WN (Wide Neck)	35	
	Standard Plus 3.3 NN (Narrow Neck)	35	
Zimmer Dental	Tapered Screw-Vent 3.5, 4.5, 5.7 / Tapered Screw-Vent 3.5, 4.5, 5.7 friction fit	30	1.25 mm Hex screwdriver
	Screw-Vent 3.3, 3.7, 4.5 / Screw-Vent 3.3, 3.7, 4.5 friction fit	30	
	SwissPlus™ 4.8	35	

The torque recommendations are a reference for your convenience and can change without further notice in order to follow manufacturer's recommendations. All trademarks and company names are the property of their respective owners.

Medical and legal disclaimer

The information (including torque recommendations) is provided for information purposes only and does not constitute advice for medical or treatment purposes, nor a legal contract, between DENTSPLY Implants and any person or entity. Although all reasonable effort is made to present accurate information (including torque recommendations), DENTSPLY Implants makes no guarantees of any kind, including but not limited to that it may be outdated, invalid or subject to debate. The information is provided "as is" without warranty of any kind, either express or implied, including without limitation, the implied warranties of merchantability, fitness for use of a particular purpose, or non-infringement.

You agree that neither DENTSPLY Implants, nor any other company within the DENTSPLY international group, will be liable to you for any loss or damages, either actual or consequential, arising out of or relating to these terms, or to your (or any third party's) reliance upon information (including torque recommendations) obtained from DENTSPLY Implants. In particular, DENTSPLY Implants will have no liability for any consequential, indirect, punitive, special or incidental damages, whether foreseeable or unforeseeable, arising out of or relating to these terms, or to your (or any third party's) reliance upon information (including torque recommendations) obtained from DENTSPLY Implants, whether based in contract, tort, statutory or other law, except only in the case of death or personal injury where and only to the extent that applicable law requires such liability.